10 Kids' Facemasks

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(10) Kids' facemasks. Single use, do not reuse mask. **Directions:** Only an adult should open the package and take out the mask. Carefully help your child wear the mask by slipping one loop over one ear, and the next loop over child's opposite ear. Adjust as needed to make comfortable. Mask should cover nose and mouth. Bend nose clip so it fits to your child's nose bridge. If your child has skin allergies, please consult a physician prior to use. Product is not suitable for toxic chemical or gas protection. **WARNING:** May be a possible choking hazard. Keep out of reach of children unless under adult supervision. If the product becomes uncomfortable or irritating, please stop using the product immediately. If the irritation continues, bring this label to a physician and seek medical advice. Do not use mask in a medical setting or where significant exposure to liquid, bodily or other hazardous fluids may be expected, in a clinical setting where the infection risk level through inhalation exposure is high, or in the presence of a high intensity heat source or flammable gas. Not intended to be used as a toy. Not intended for use by children less than 4 years of age. Materials: 3-layer non-woven fabric.

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