# Nūby™ Nasal Aspirator

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**CLEANING:** After each use, unscrew the bulb to separate it from the nozzle. Wash all parts in warm, soapy water. Rinse and dry thoroughly. Can be boiled; microwave steam sterilizer safe. **DO NOT** microwave the filters. **WARNING:** Do not push the nozzle into your baby's nostril. Suck through the mouthpiece, but never blow into it. Aspirator is to be used in the nose only, not the mouth or ears. This product is NOT intended for use by children and is intended to be used as a parental aid only, as it may present a small parts hazard to children. Never leave a child unattended while using this or any other infant product. Please retain this packaging for future reference. **USE:** Before first use, wash well and make sure the filter is fitted in the middle chamber.

1. If the mucus is very dry, try moistening it with a saline solution like nasal drops or spray.
2. Put the nozzle at the entrance of your baby's nostril (but not inside), right next to any visible mucus.
3. Suck through the mouthpiece. You can vary the pressure, and it is not possible to over-suck. Never blow into the mouthpiece. The mucus collects hygienically inside the sealed chamber.

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© 2016 Nuby™ is a trademark licensed to Luv n' care®, Ltd.
Made in China to Luv n' care® specifications. Manufactured by Luv n' care®, Ltd.