## Nasal Aspirator & Ear Syringe

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**EN Nasal aspirator and ear syringe:** INSTRUCTIONS FOR USE: 1) Nasal Aspirator: first squeeze the bulb portion of the aspirator, then place the end gently against baby's nostril and release the pressure on the bulb. Clear any matter from the tip after each suction. **WARNING:** DO NOT squeeze bulb while tip is against baby's nostril. 2) Ear Syringe Aspirator: Simply remove the nasal tip and replace with the extension and flexible ear syringe tip. To remove debris from baby's ear, squeeze bulb, place flexible extension in the outer ear canal and slowly release pressure on the aspirator bulb. Use of a wax removal solution and an ear syringe is the only approved method for the removal of excess wax. **WARNING:** DO NOT squeeze bulb while tip is against ear canal. Never use cotton swabs to remove wax and debris from baby's ear canal. **For your child's safety and health:** **WARNING:** This product is not a toy. Always use this product with adult supervision. Before each use, inspect the product. Throw away at the first sign of damage or weakness. Keep all components not in use out of reach of children. **CLEANING:** Wash thoroughly before and after each use in warm, soapy water; rinse well. Keep this information.

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© 2019 Nuby is a trademark owned by or licensed to Luv n' care, Ltd. Made in China to Luv n' care, Ltd. specifications. Manufactured by Luv n' care, Ltd. Imported by New Valmar b.v.b.a., Buntstraat 104, 9940 Evergem BELGIUM - WWW.NUBYCLUB.COM